

# SAS

## SNAP-ON ABUTMENT SYSTEM

### SNAP-ON ABUTMENT SYSTEM ADVANTAGES:

- Can be used in both anterior and posterior areas of the mouth for cement-retained single-unit crowns
- Can be used as the final abutment, or provisionalized chairside using the Snap-on Temporary Cap
- Everything in one package - all necessary components to complete the restorative procedure
- Concave emergence profile for increased soft-tissue volume and higher aesthetics
- Available in 4.0mm and 5.5mm abutment heights
- Developed with ease-of-use in mind

### SNAP-ON IMPRESSION COPING


The Snap-on Impression Coping snaps onto the abutment allowing the doctor to use a closed tray impression technique. The Snap-on impression coping with its ease-of-use, allows for an accurate and quick impression.

### SNAP-ON TEMPORARY CAP


The Snap-on Temporary Cap is made from PEEK and can be used as a healing cap, or as a base for a temporary crown.

### SNAP-ON ANALOG

The Snap-on Analog securely snaps into the impression coping that is in the impression. The Analog may be scanned by an optical scanner in the laboratory for the manufacture of a CAD/CAM crown.

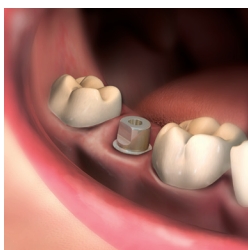


The abutment is placed into the implant and the retaining screw is torqued into place. (see the torque chart on the page below)

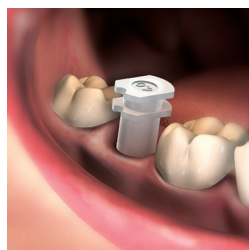

The Snap-on impression coping snaps onto the abutment.



The Snap-on impression coping is picked up in the impression with a close tray impression technique.

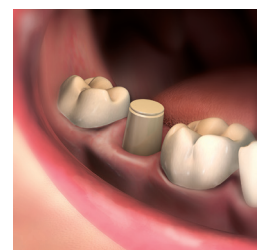


The Snap-on Temporary Cap is placed onto the Snap-on Abutment for the duration of the healing phase, or can be provisionalized with a temporary crown.



NARROW PLATFORM			
Cat NO.	Ø (mm)	L (mm)	H (mm)
44-70021	4.5	4.0	1
44-70022	4.5	4.0	2
44-70023	4.5	4.0	3
44-70024	4.5	4.0	4
44-70025	4.5	4.0	5
44-70026	4.5	5.5	1
44-70027	4.5	5.5	2
44-70028	4.5	5.5	3
44-70029	4.5	5.5	4
44-70030	4.5	5.5	5


STANDARD PLATFORM			
Cat NO.	Ø (mm)	L (mm)	H (mm)
44-70001	4.5	4.0	1
44-70002	4.5	4.0	2
44-70003	4.5	4.0	3
44-70004	4.5	4.0	4
44-70005	4.5	4.0	5
44-70006	4.5	5.5	1
44-70007	4.5	5.5	2
44-70008	4.5	5.5	3
44-70009	4.5	5.5	4
44-70010	4.5	5.5	5
44-70011	6.0	4.0	1
44-70012	6.0	4.0	2
44-70013	6.0	4.0	3
44-70014	6.0	4.0	4
44-70015	6.0	4.0	5
44-70016	6.0	5.5	1
44-70017	6.0	5.5	2
44-70018	6.0	5.5	3
44-70019	6.0	5.5	4
44-70020	6.0	5.5	5

WIDE PLATFORM			
Cat NO.	Ø (mm)	L (mm)	H (mm)
44-70041	6.0	4.0	1
44-70042	6.0	4.0	2
44-70043	6.0	4.0	3
44-70044	6.0	5.5	1
44-70045	6.0	5.5	2
44-70046	6.0	5.5	3

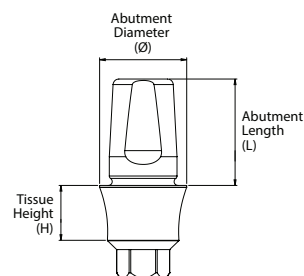

CONICAL PLATFORM			
Cat NO.	Ø (mm)	L (mm)	H (mm)
44-72441	4.5	4.0	1
44-72442	4.5	4.0	2
44-72443	4.5	4.0	3
44-72444	4.5	4.0	4
44-72445	4.5	4.0	5
44-72461	6.0	4.0	1
44-72462	6.0	4.0	2
44-72463	6.0	4.0	3
44-72464	6.0	4.0	4
44-72465	6.0	4.0	5

CONICAL PLATFORM			
Cat NO.	Ø (mm)	L (mm)	H (mm)
44-72541	4.5	5.5	1
44-72542	4.5	5.5	2
44-72543	4.5	5.5	3
44-72544	4.5	5.5	4
44-72545	4.5	5.5	5
44-72561	6.0	5.5	1
44-72562	6.0	5.5	2
44-72563	6.0	5.5	3
44-72564	6.0	5.5	4
44-72565	6.0	5.5	5

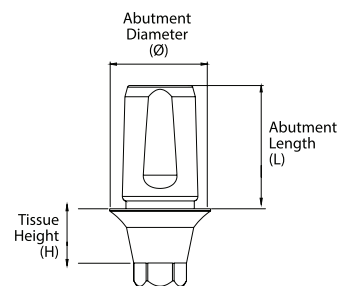

Screw Platform	Torque (N-cm)
SP / WP Abutment Screws	30-35
NP Abutment Screw	25-30
Conical Abutment Screw	25

**Keystone Dental, Inc.**  
 154 Middlesex Turnpike  
 Burlington, MA 01803 USA  
 General Inquiries: 781-328-3600  
 Customer Relations: 866-902-9272 or 781-328-3490  
 Fax: 866-903-9272 or 781-328-3400  
 Email: info@keystonedental.com